

Dentium


Osteotome Kit

Crestal approach

Osteotome Kit



Crestal approach (Sinus lifting]

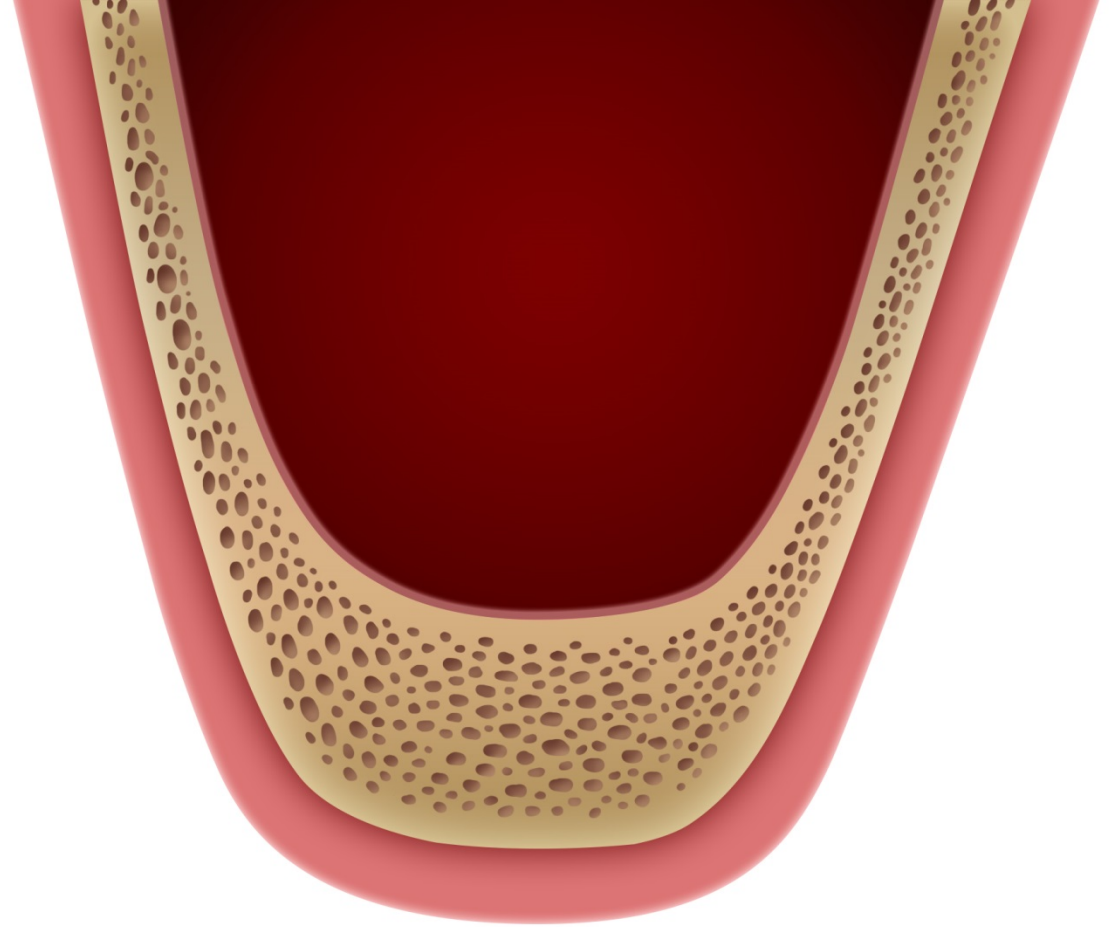


Crestal approach (Sinus lifting)

1st surgery

First guide drill

1000rpm with irrigation

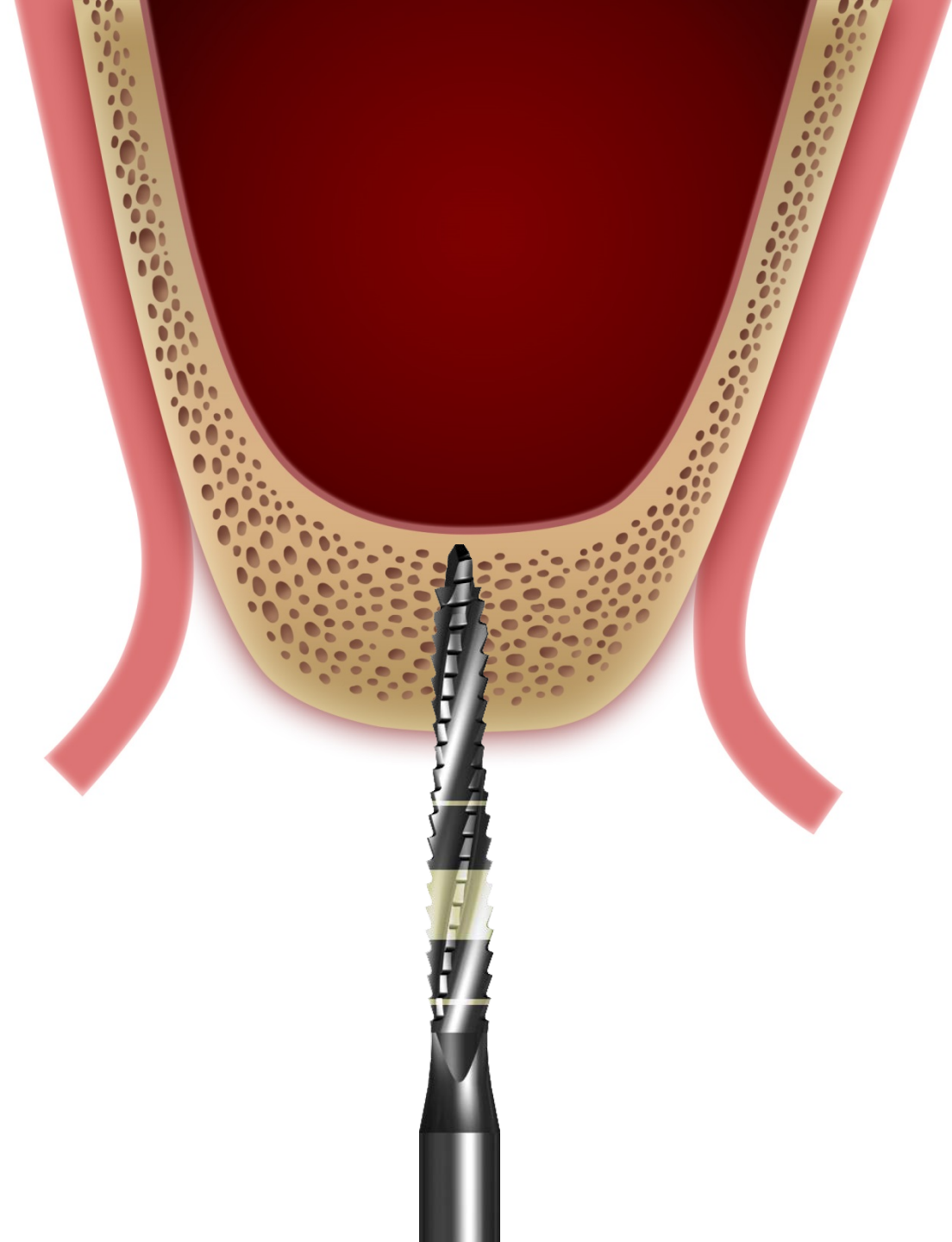


Crestal approach (Sinus lifting)

1st surgery

Ø3.4 Final drill

50rpm without irrigation

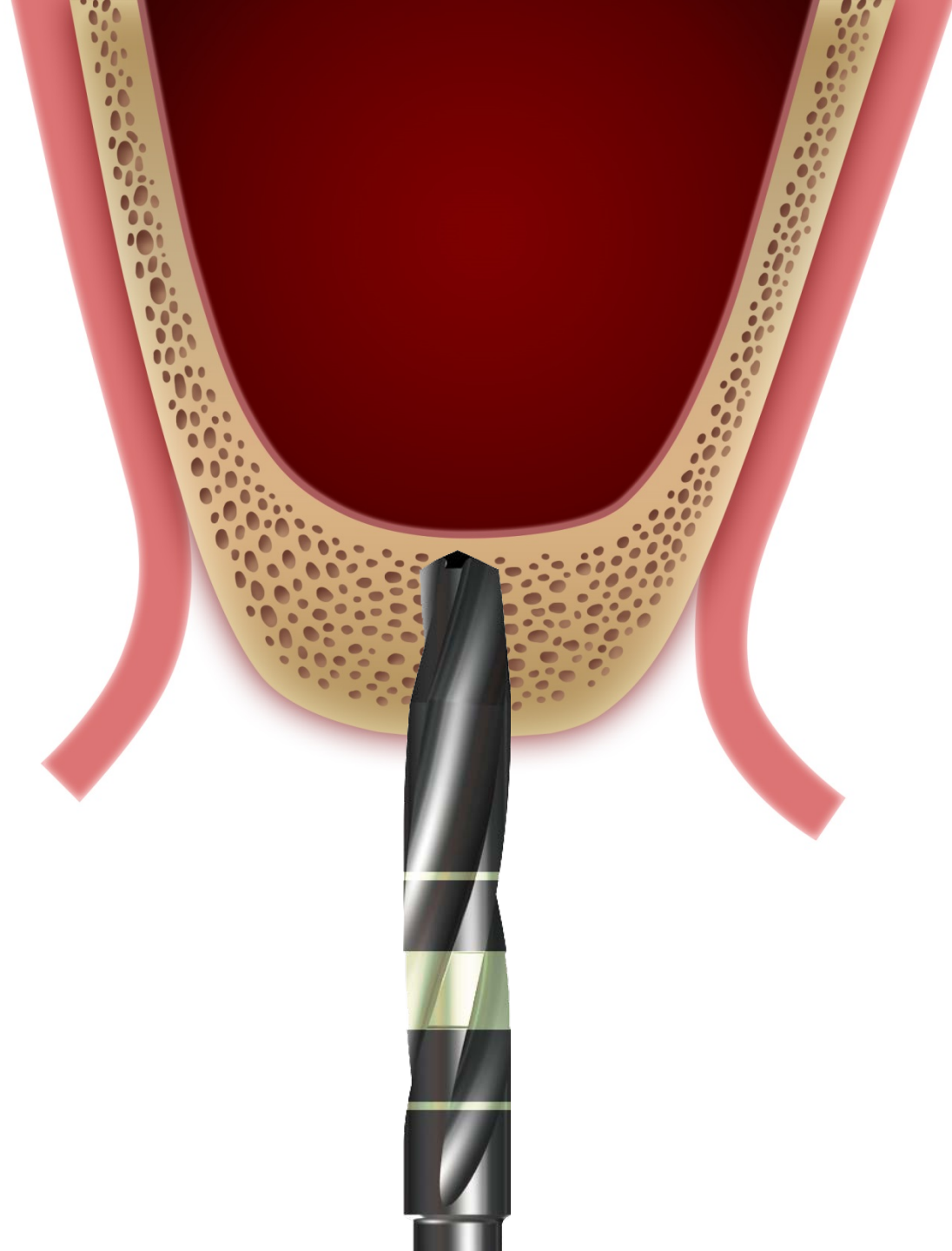


Crestal approach (Sinus lifting)

1st surgery

Ø3.8 Final drill

50rpm without irrigation

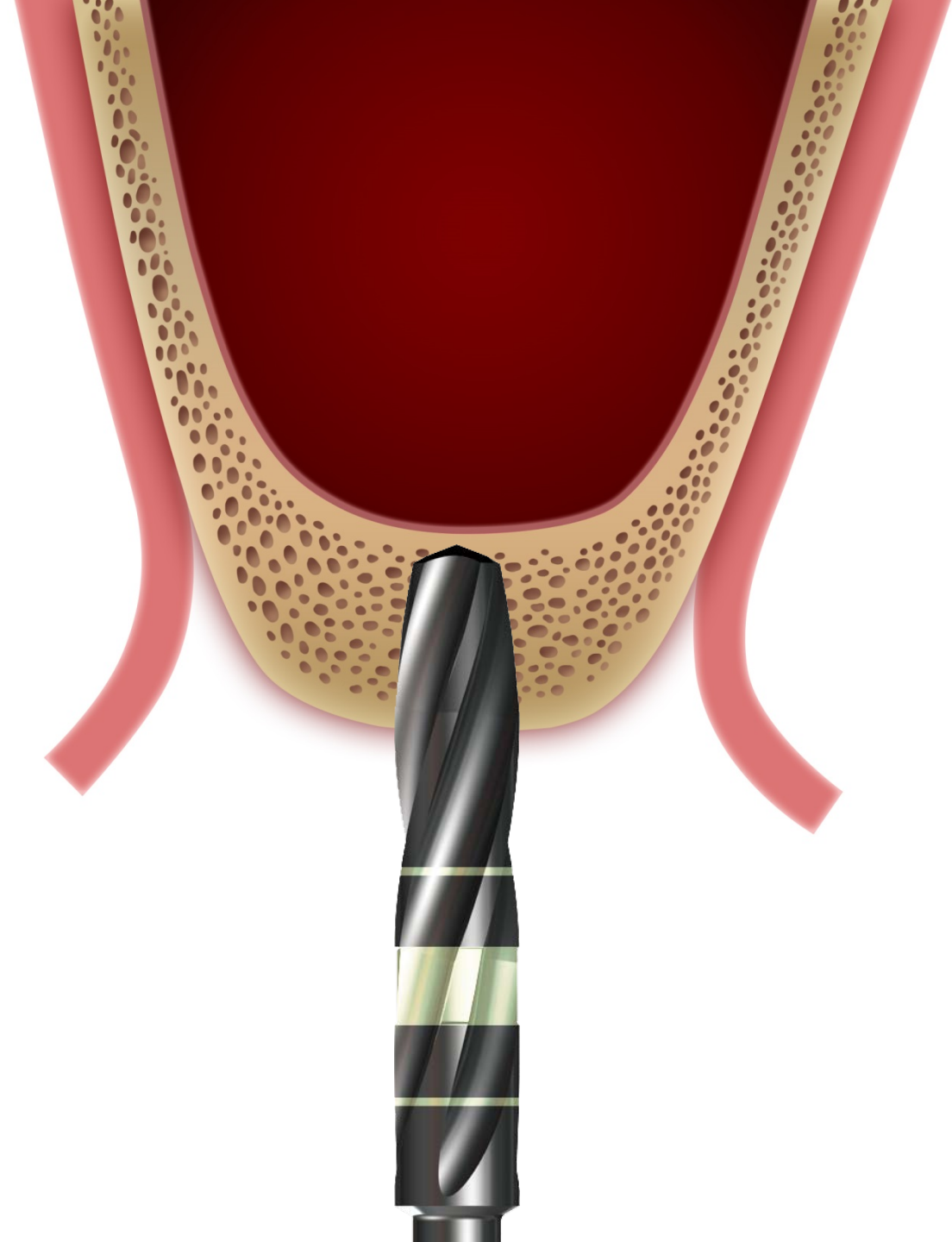


Crestal approach (Sinus lifting)

1st surgery


Ø4.3 Final drill

50rpm without irrigation

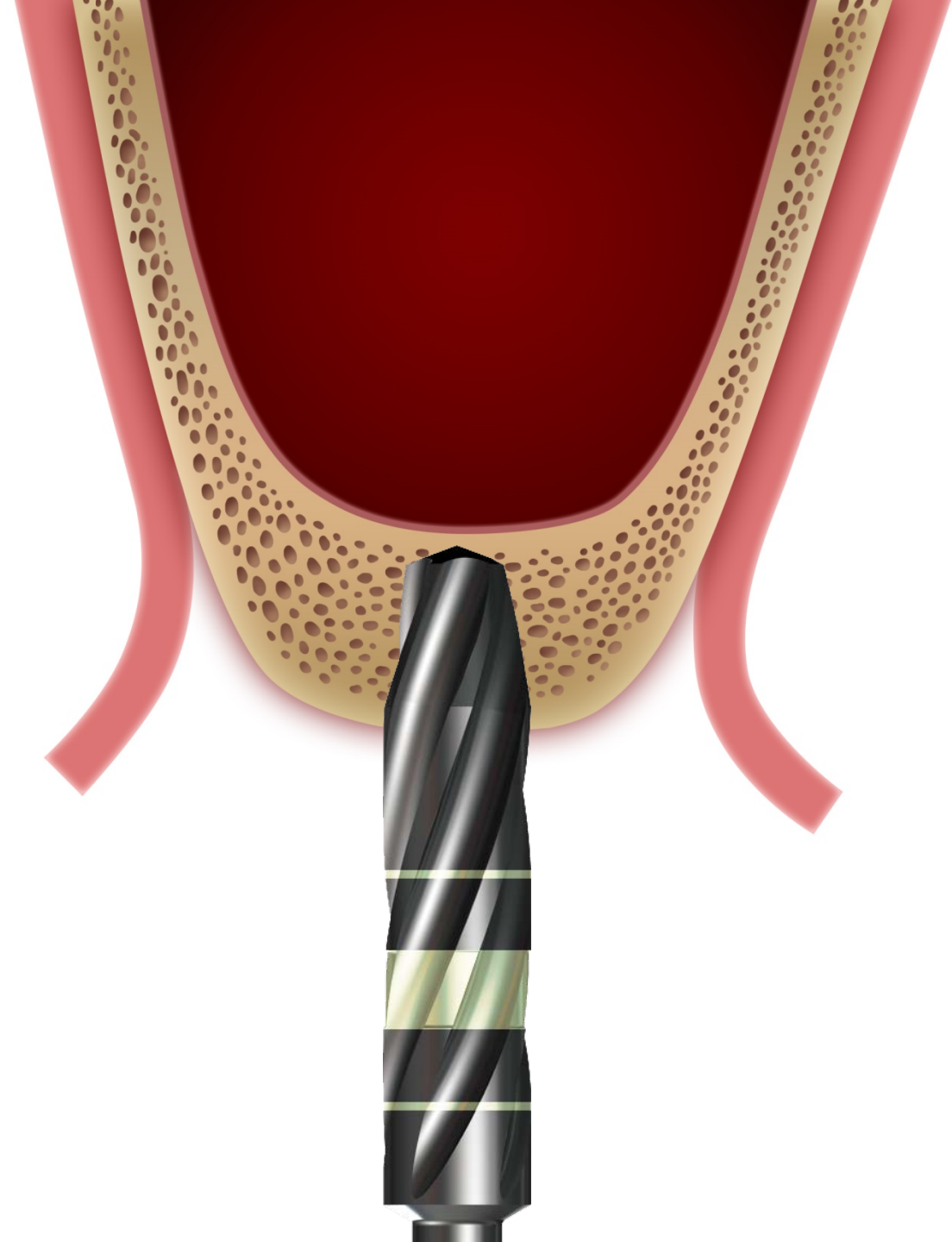


Crestal approach (Sinus lifting)

1st surgery

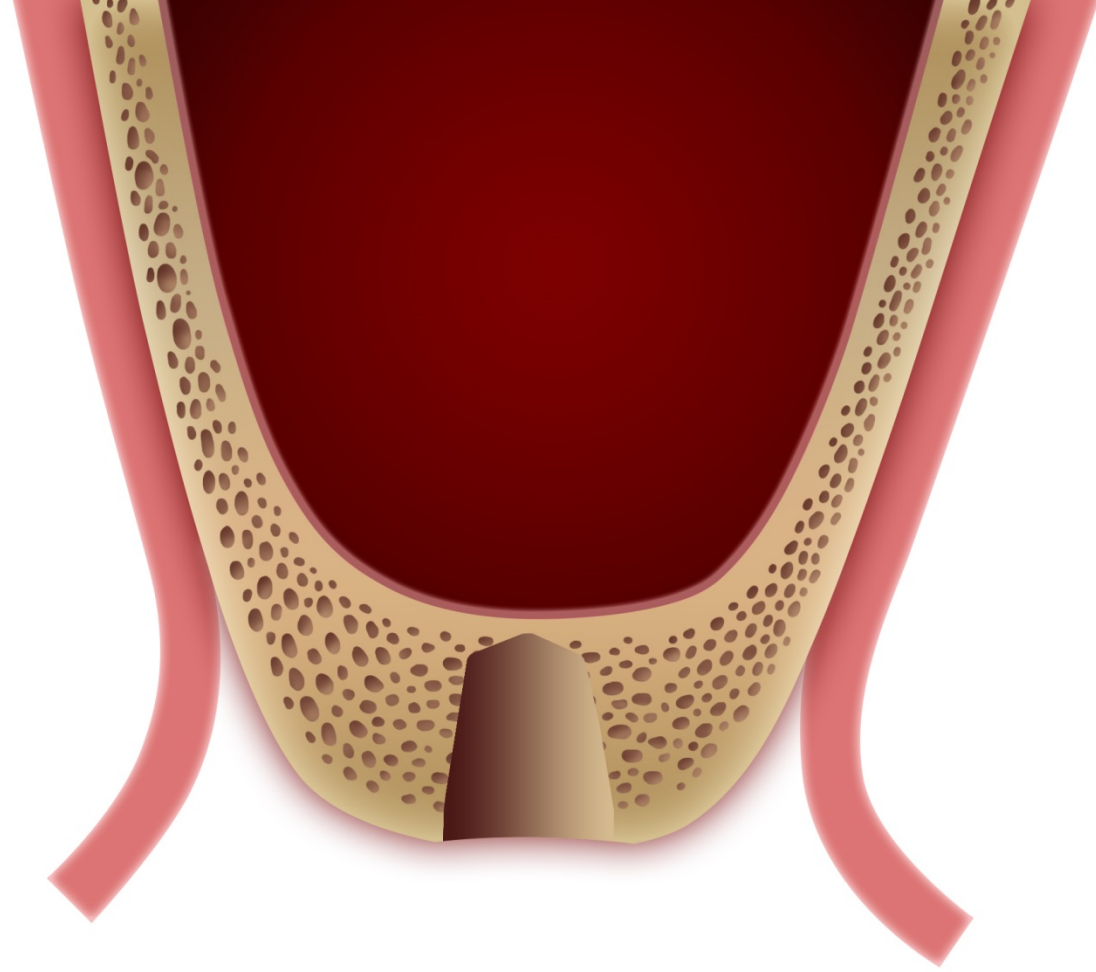

Crestal approach (Sinus lifting)


1st surgery

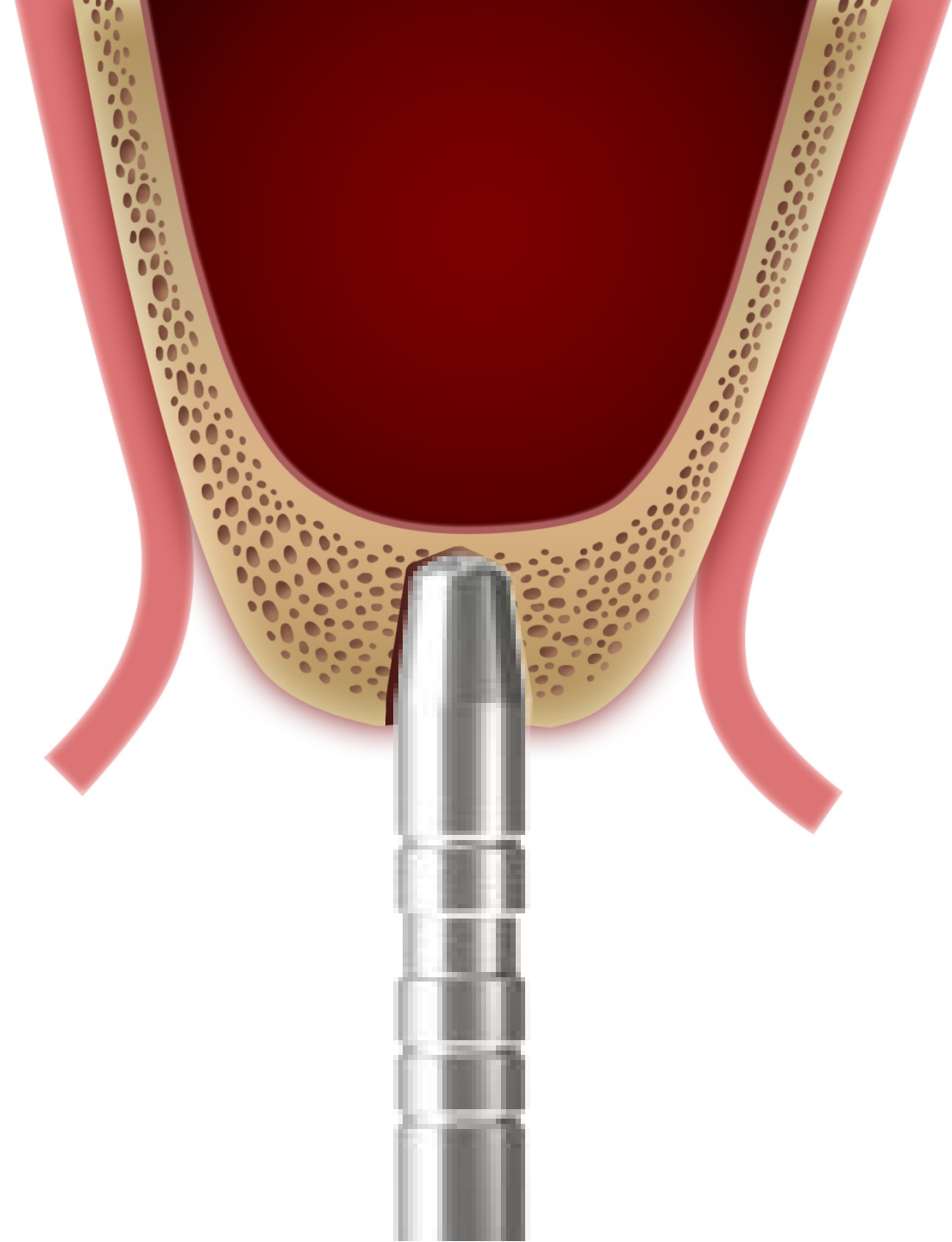

Crestal approach (Sinus lifting)

1st surgery

After Ø4.3 Final drilling, eliminate the residual bone [1mm] using a **Osteotome** [in hard bone] until you feel a slight drop.

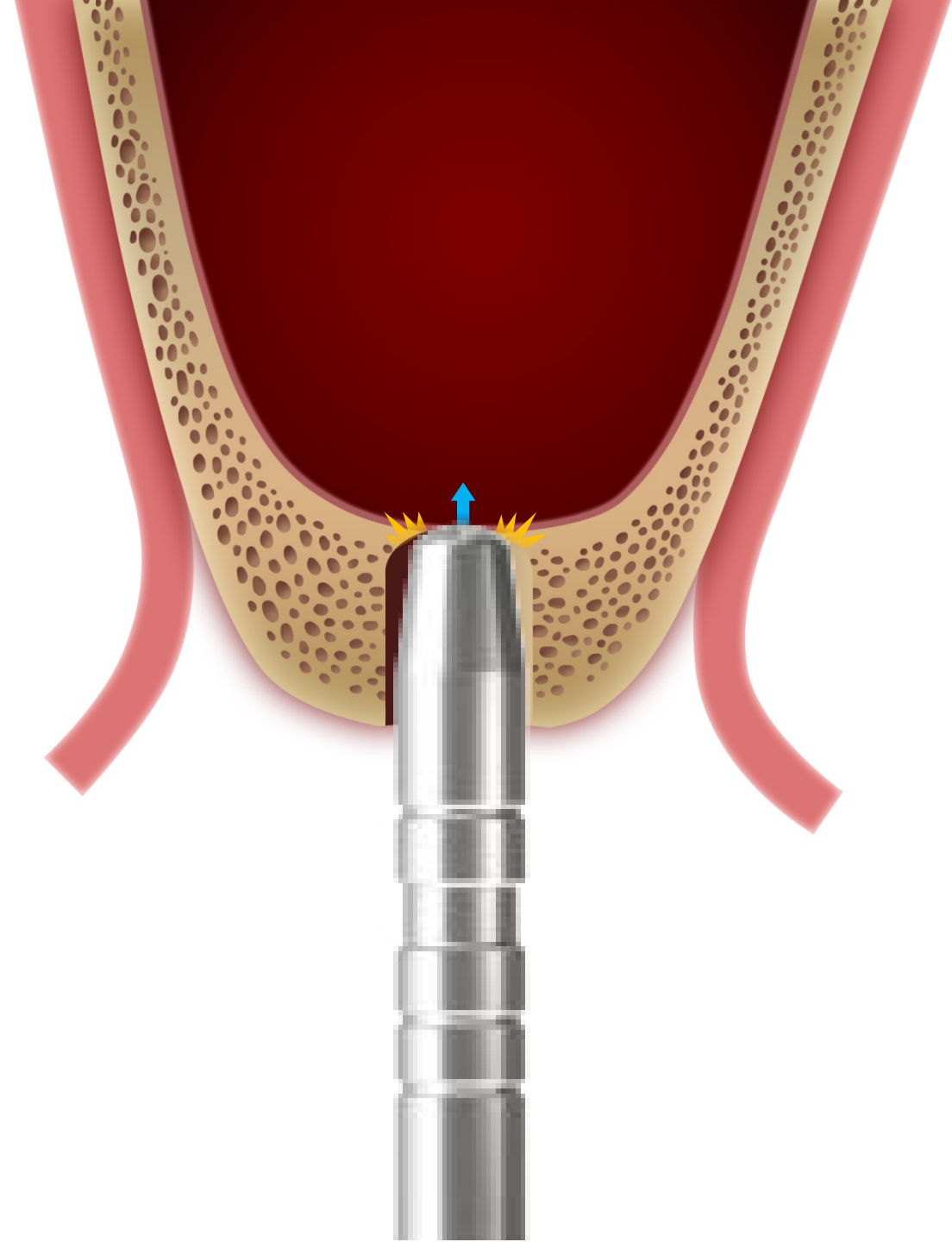

Osteotome Kit

Final drill type / Scale 1: 0.4 / mm



XOF20A1



XOF34A1



XOF38A1



XOF43A1



XOF48A1