


Dentium


Surgical Guide (Polymer Guide)

NEW


## Drilling a hole in stone model

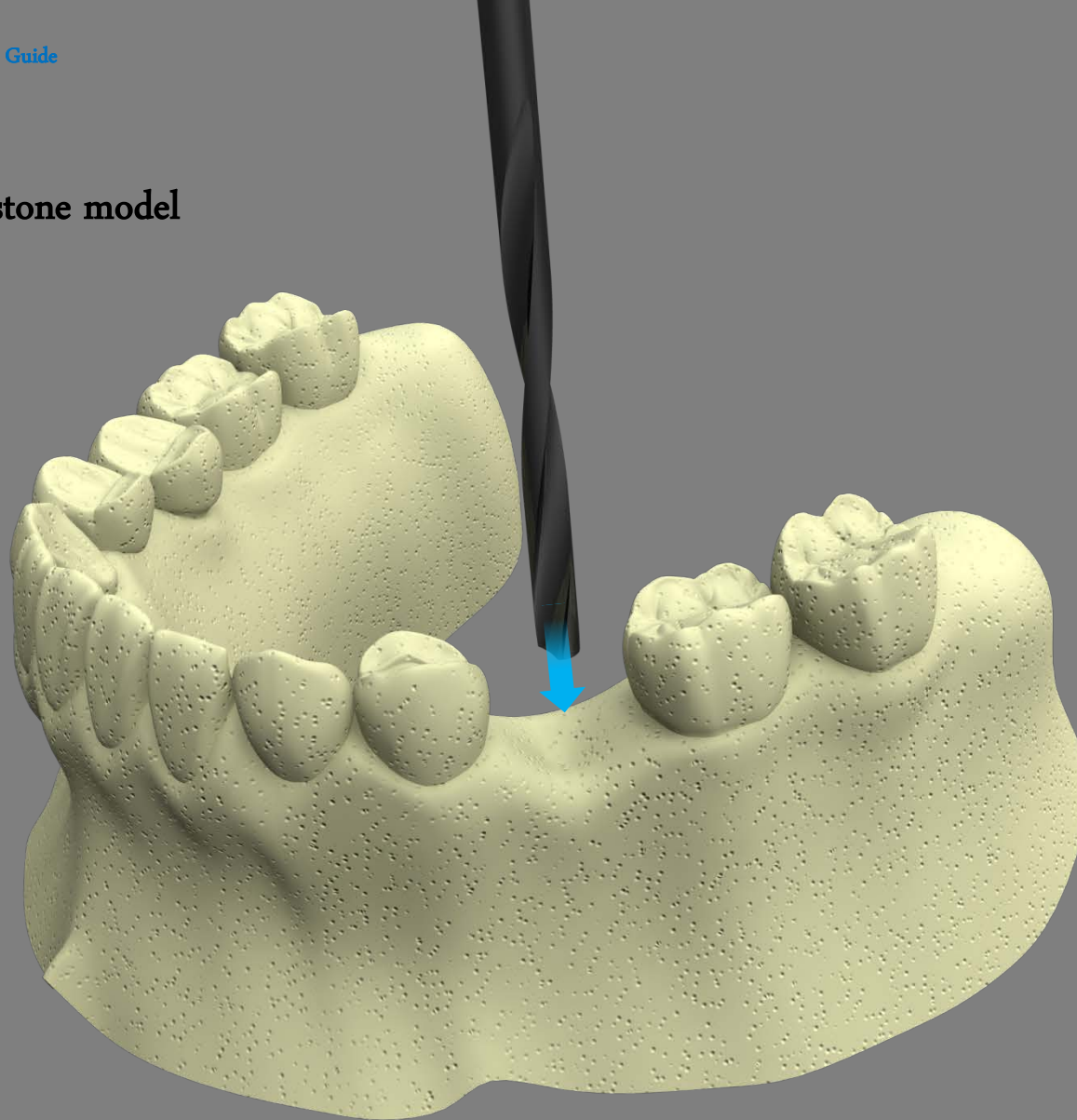


## Drilling a hole in stone model

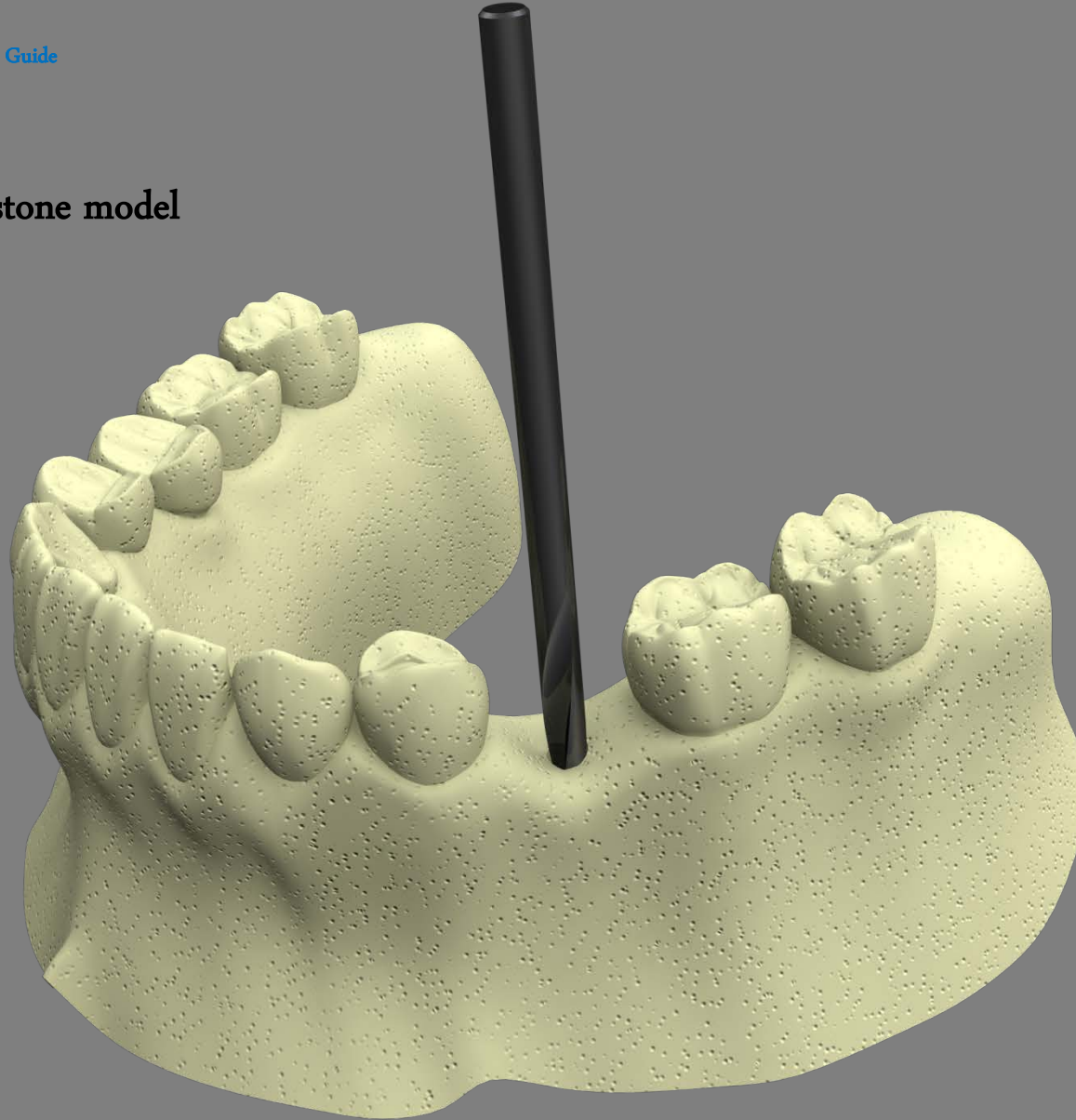

## Drilling a hole in stone model

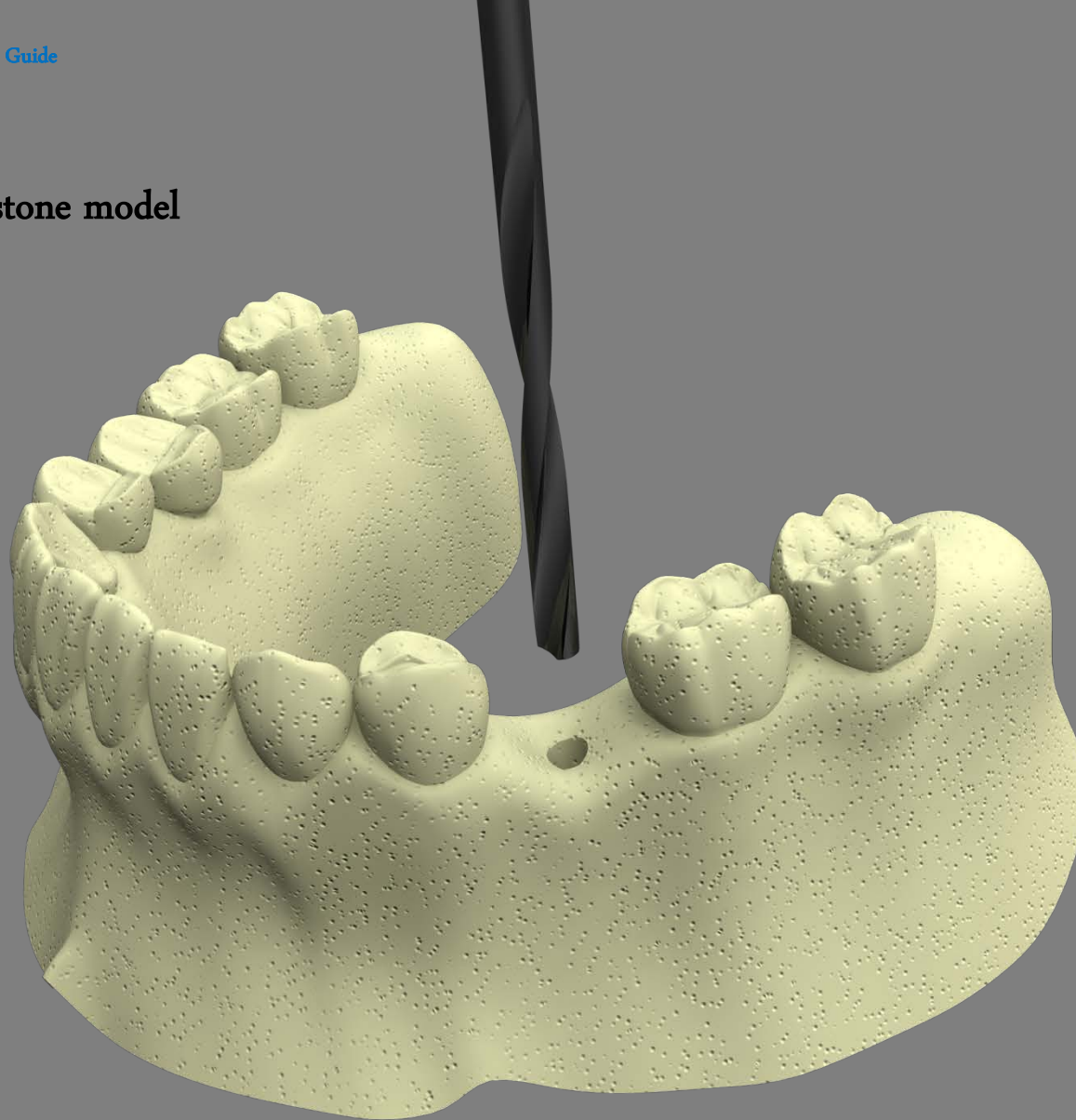


## Insertion of Guide Pin

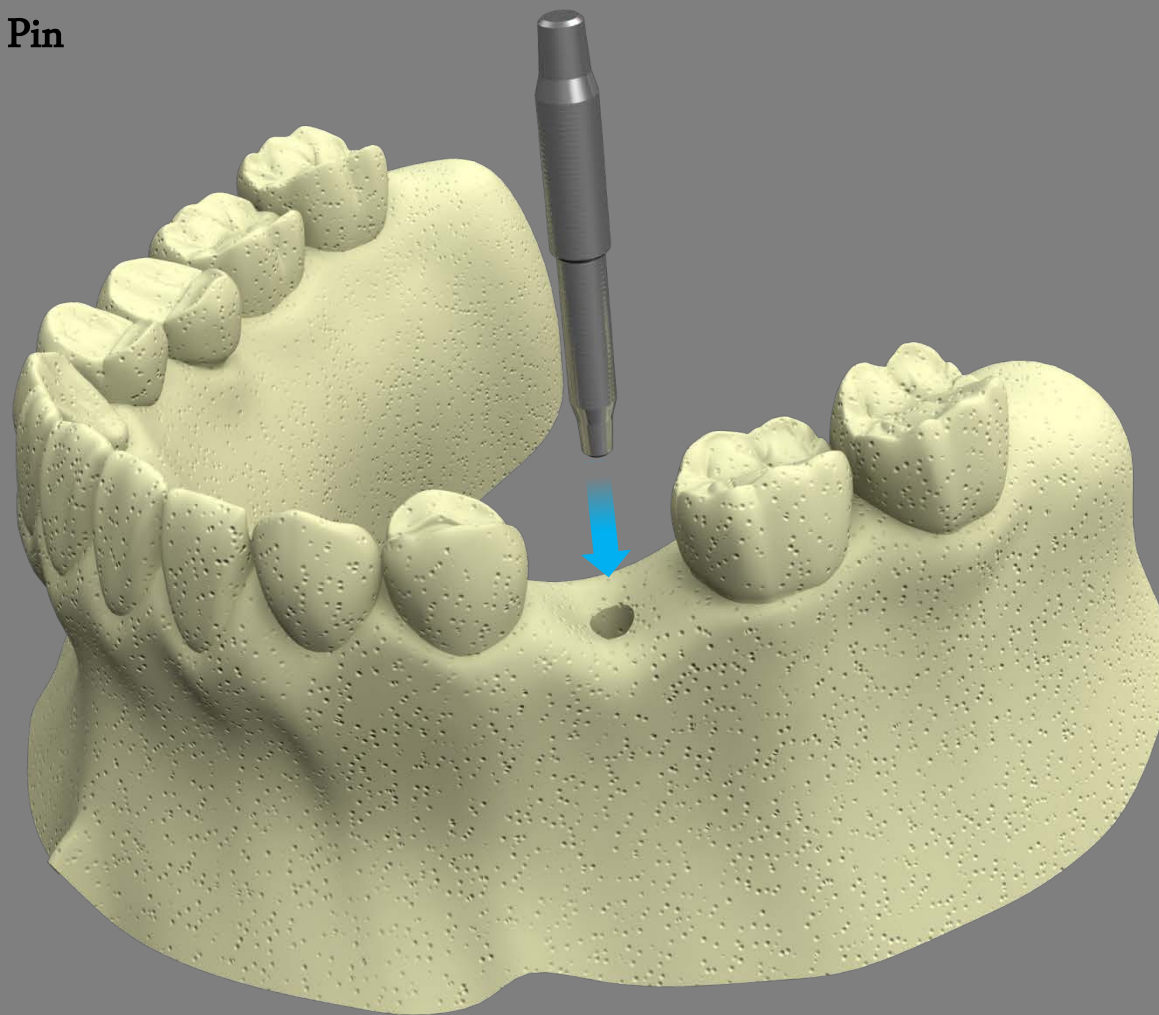



## Insertion of Guide Pin

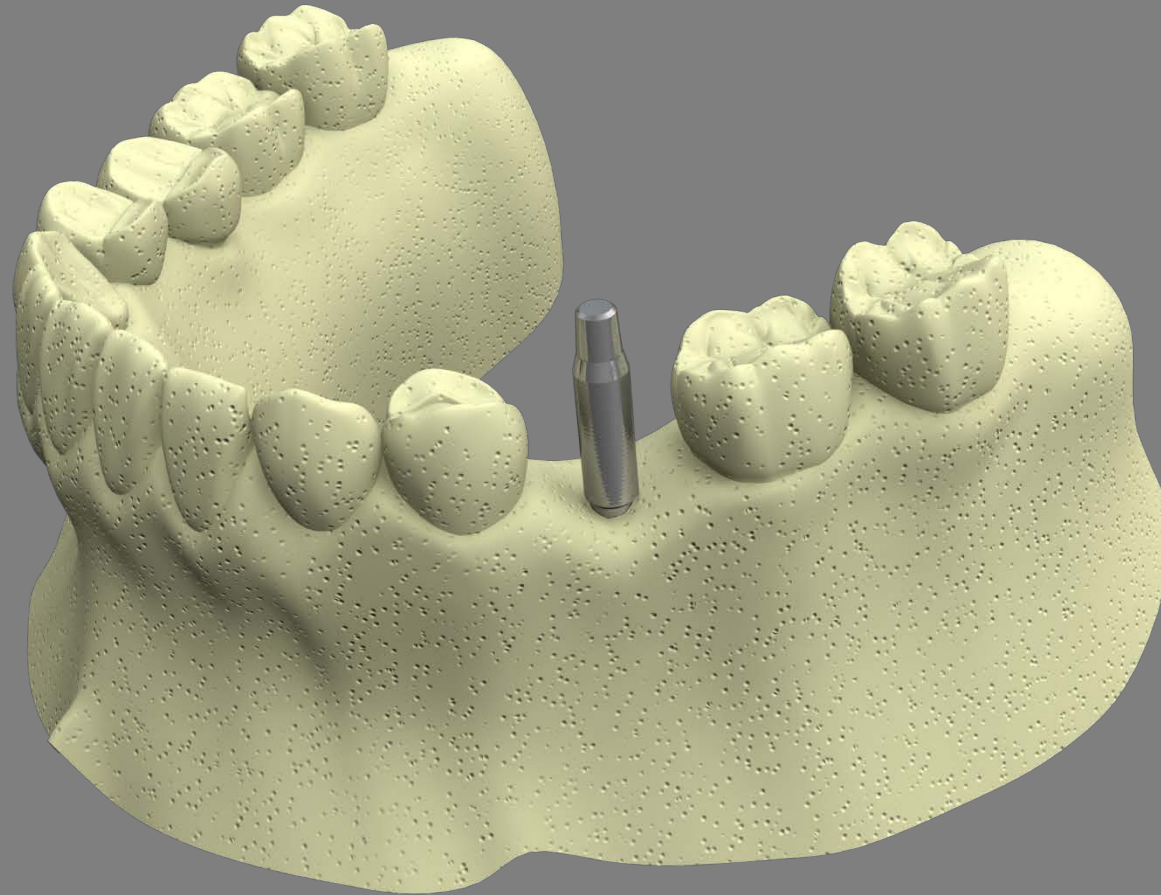

**Making precise surgical guide  
in minutes using hot water**

More than 65°C



**Making precise surgical guide  
in minutes using hot water**

More than 65°C






**Making precise surgical guide  
in minutes using hot water**


More than 65°C




## Application of Polymer Guide in stone model

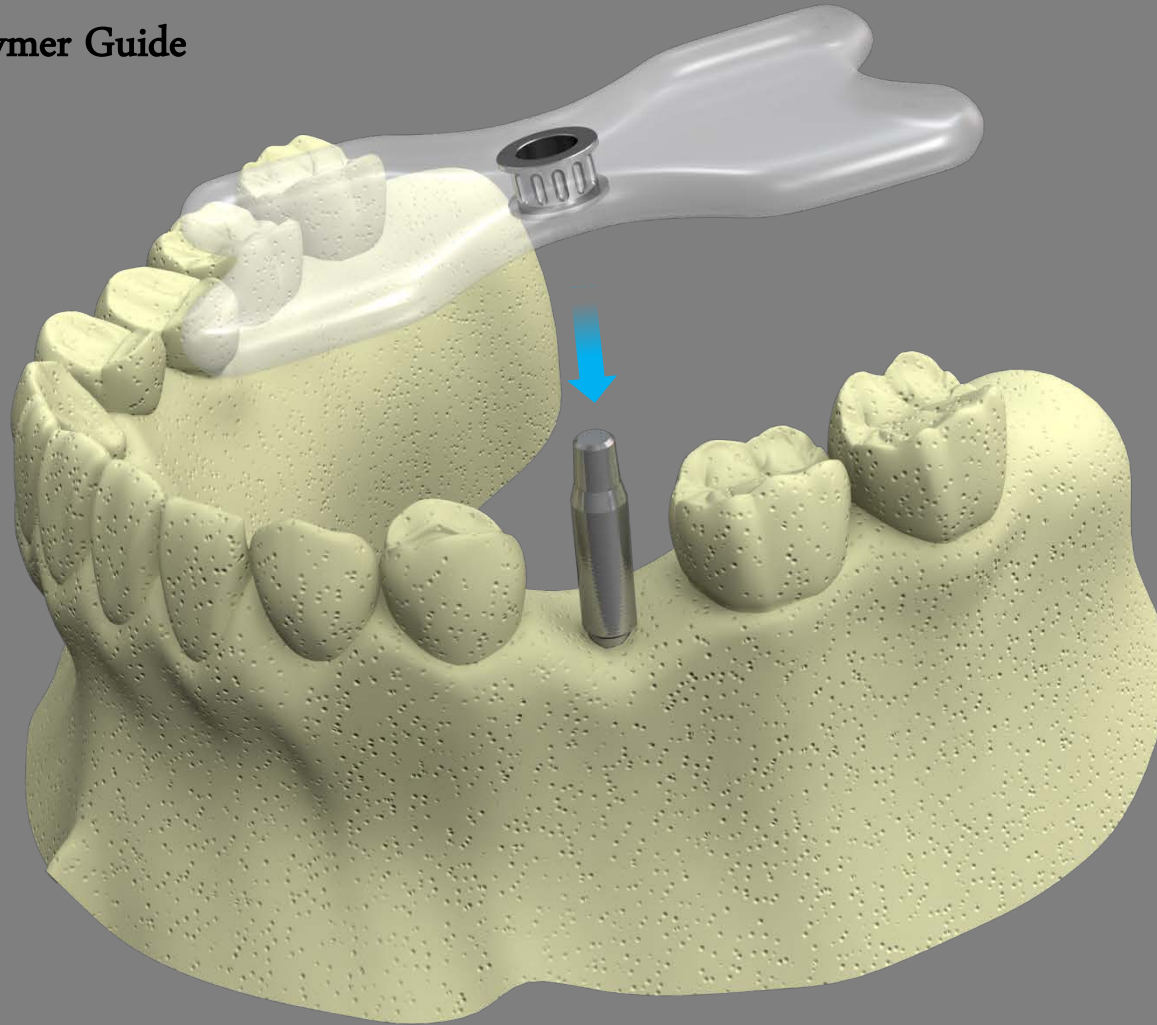


## Application of Polymer Guide in stone model

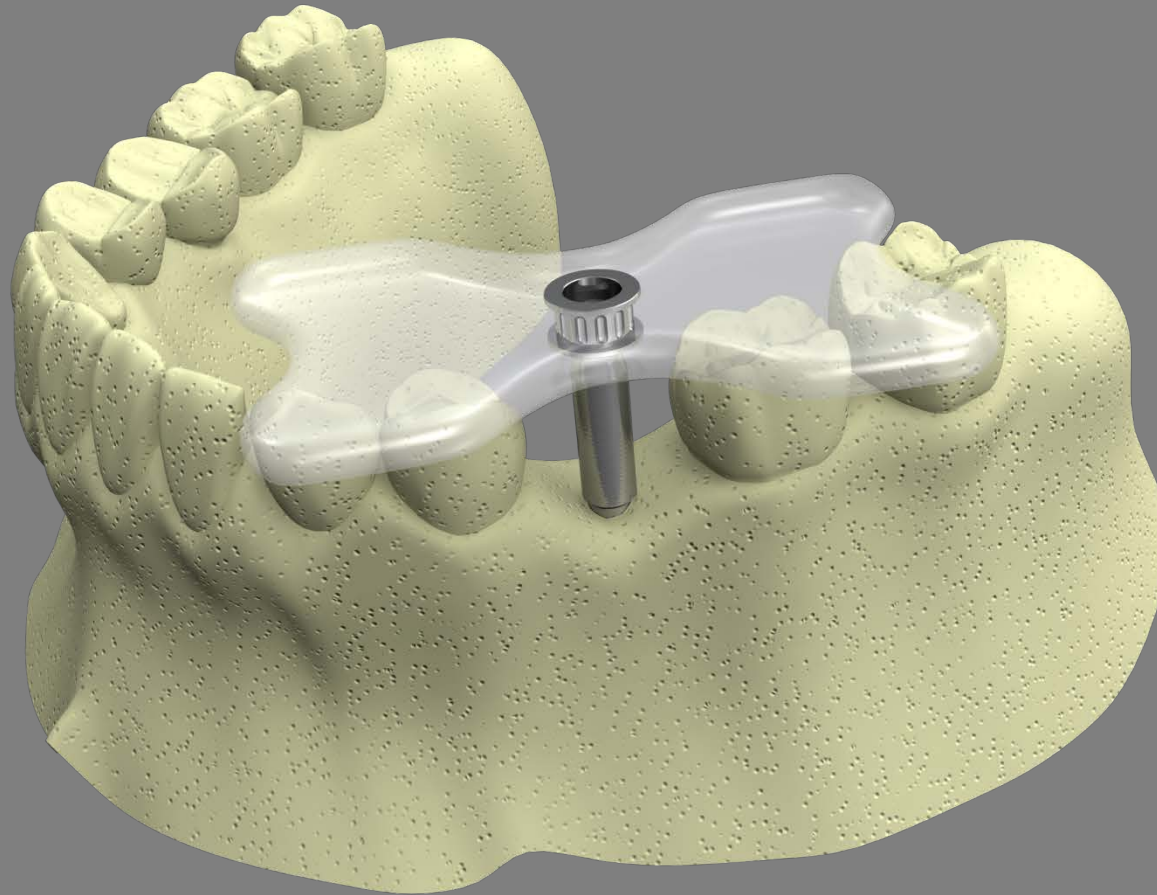

## Application of Polymer Guide in stone model

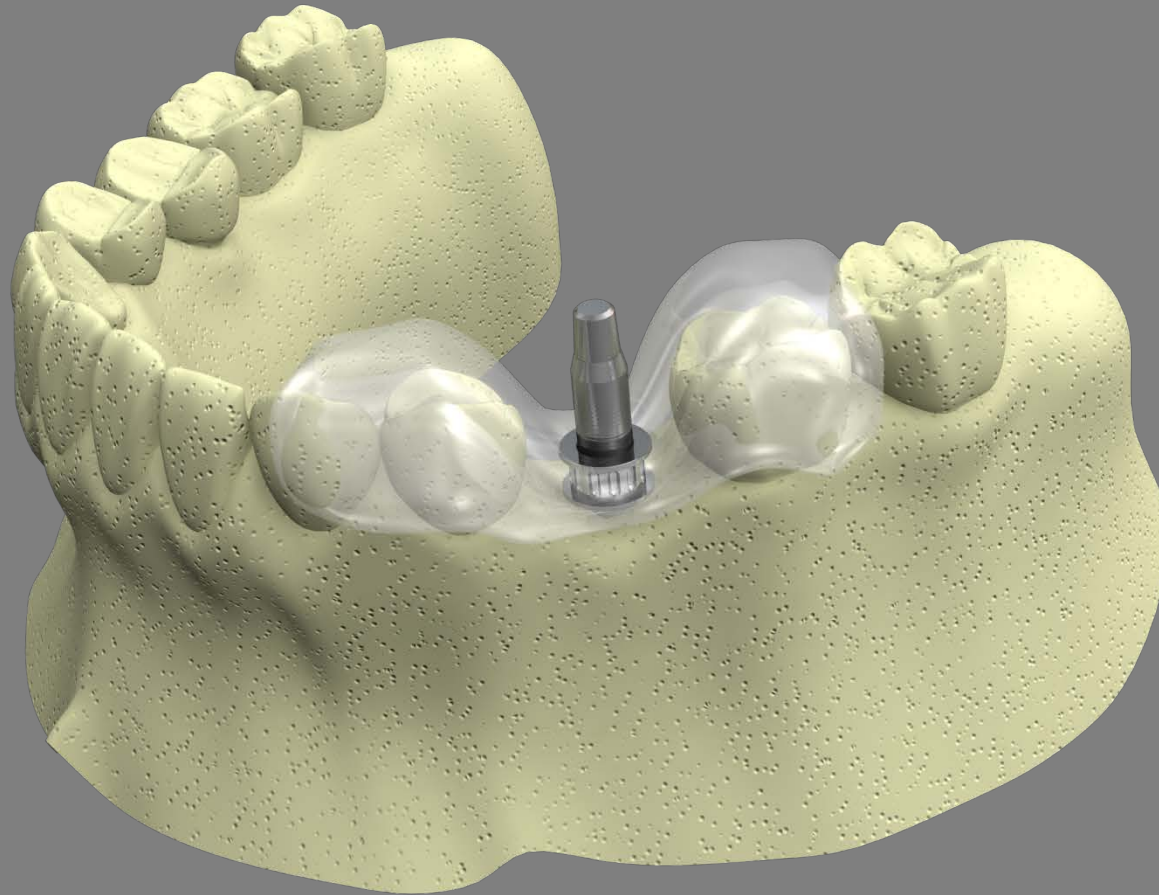


## Application of Polymer Guide in stone model

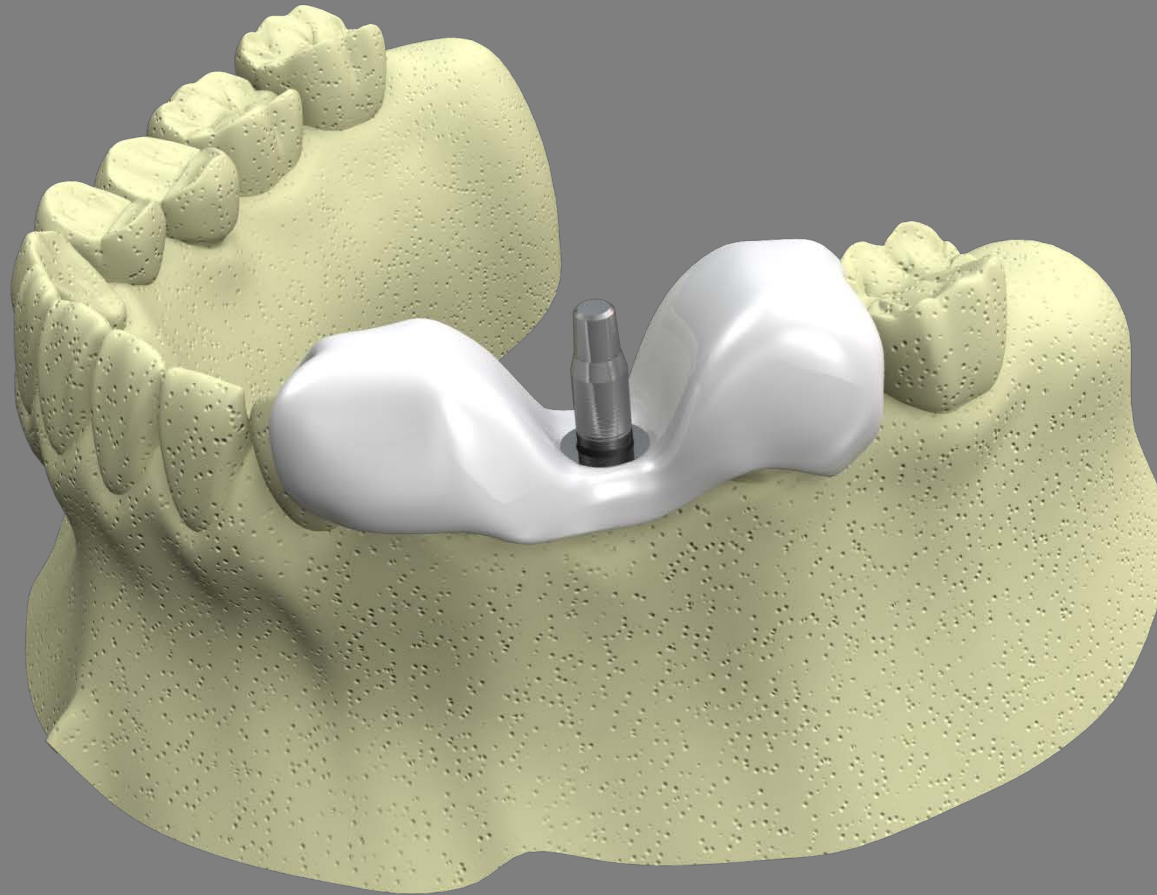





## Removal of Guide Pin

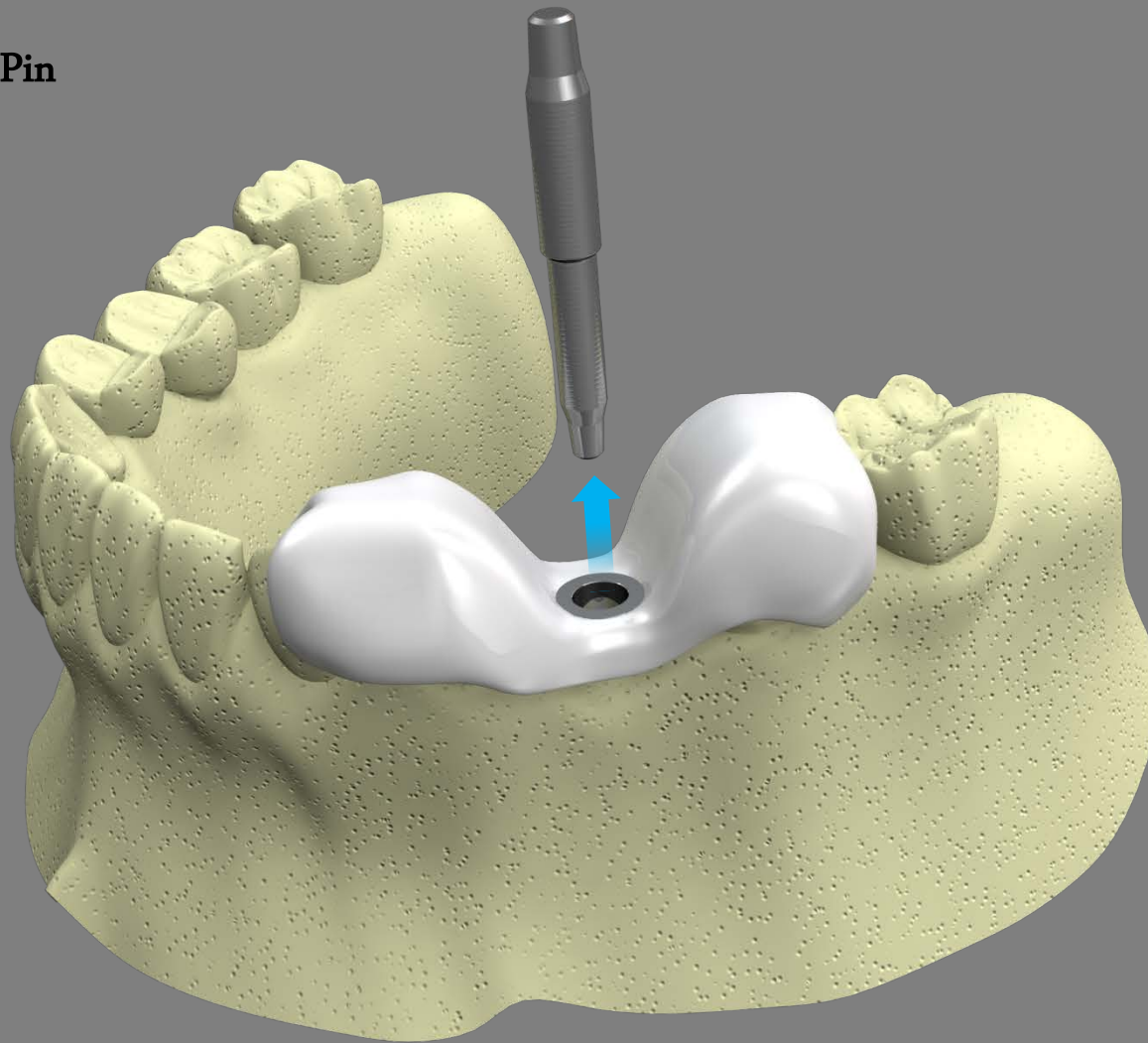




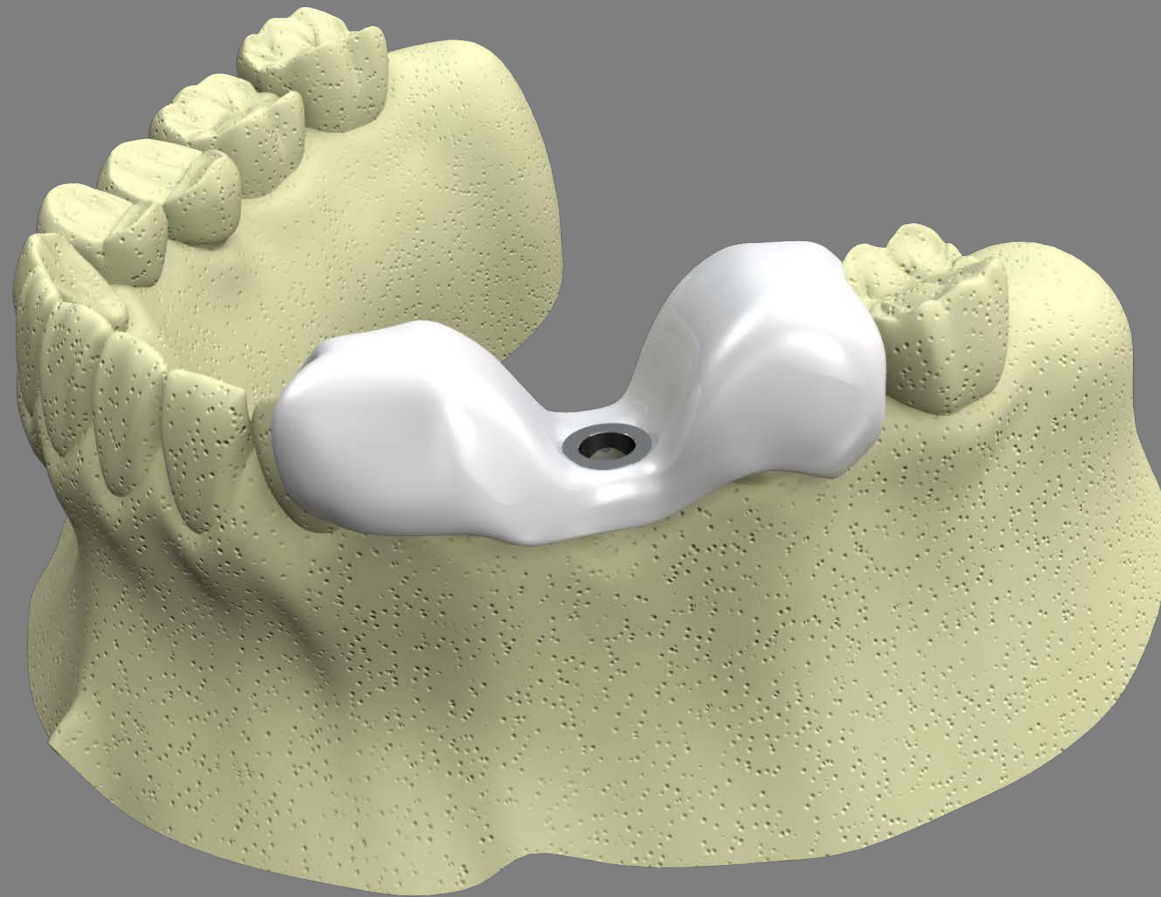


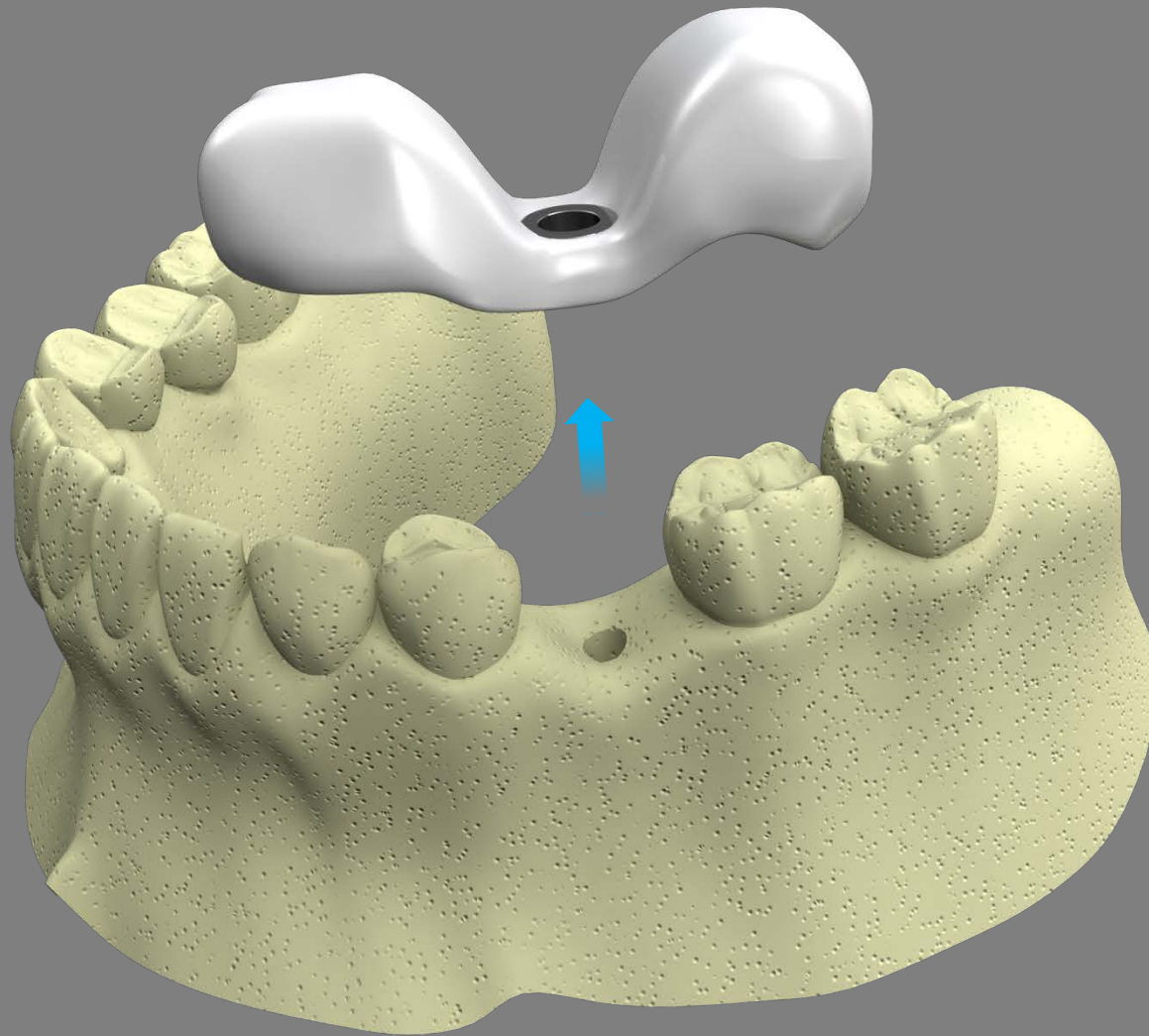


**Position the Polymer Guide  
in the mouth**

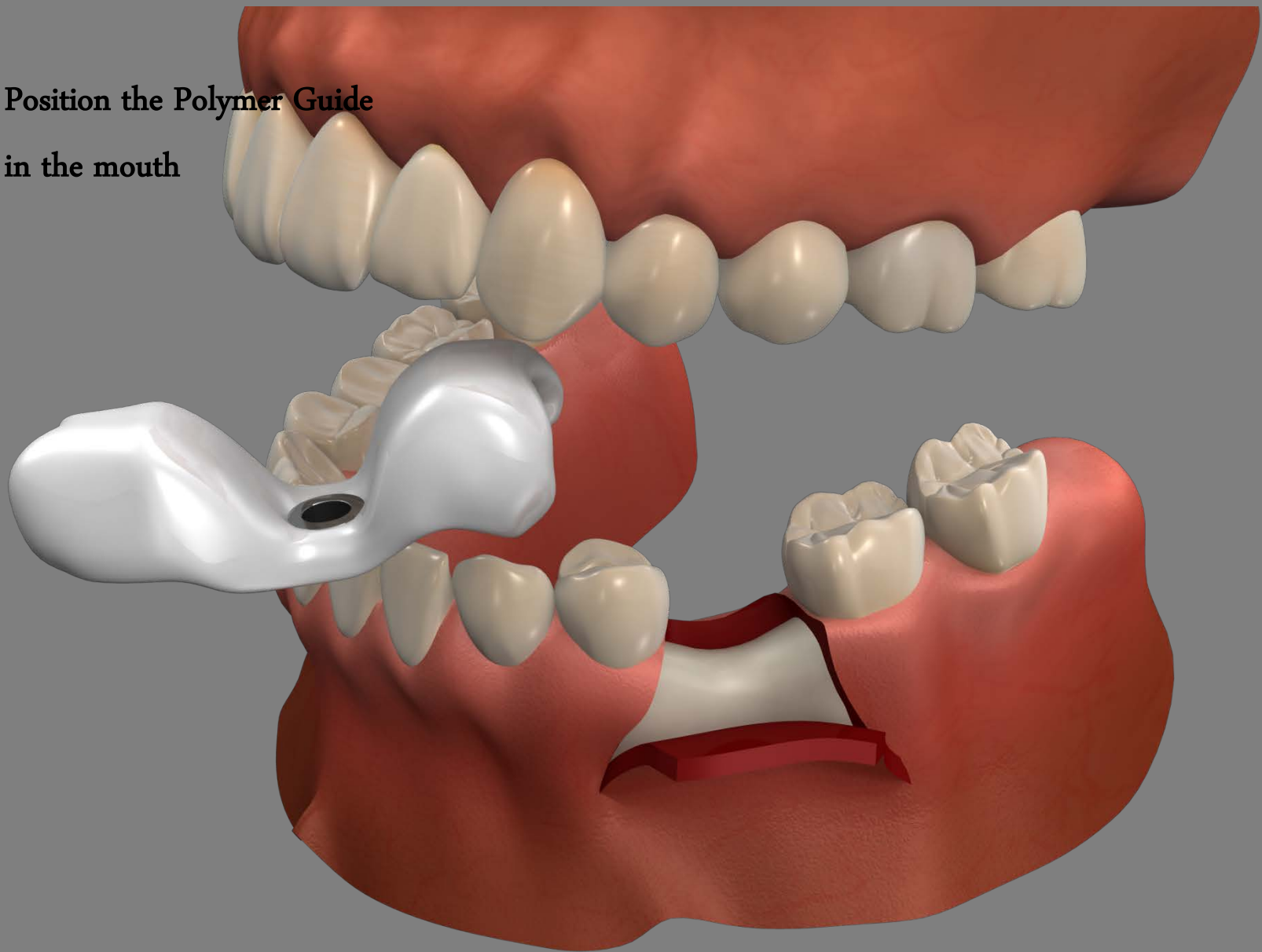

**Position the Polymer Guide  
in the mouth**

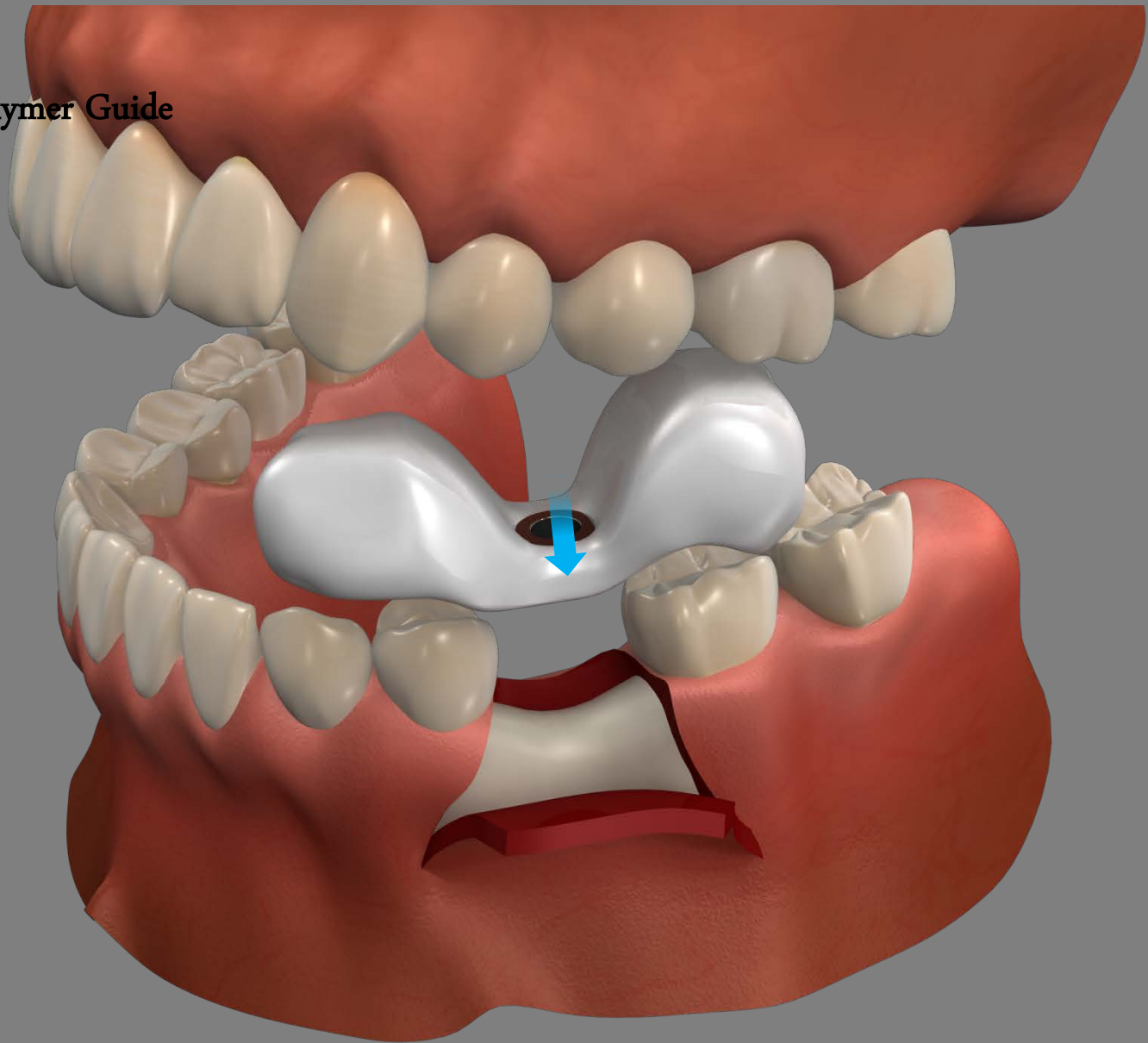


**Position the Polymer Guide  
in the mouth**

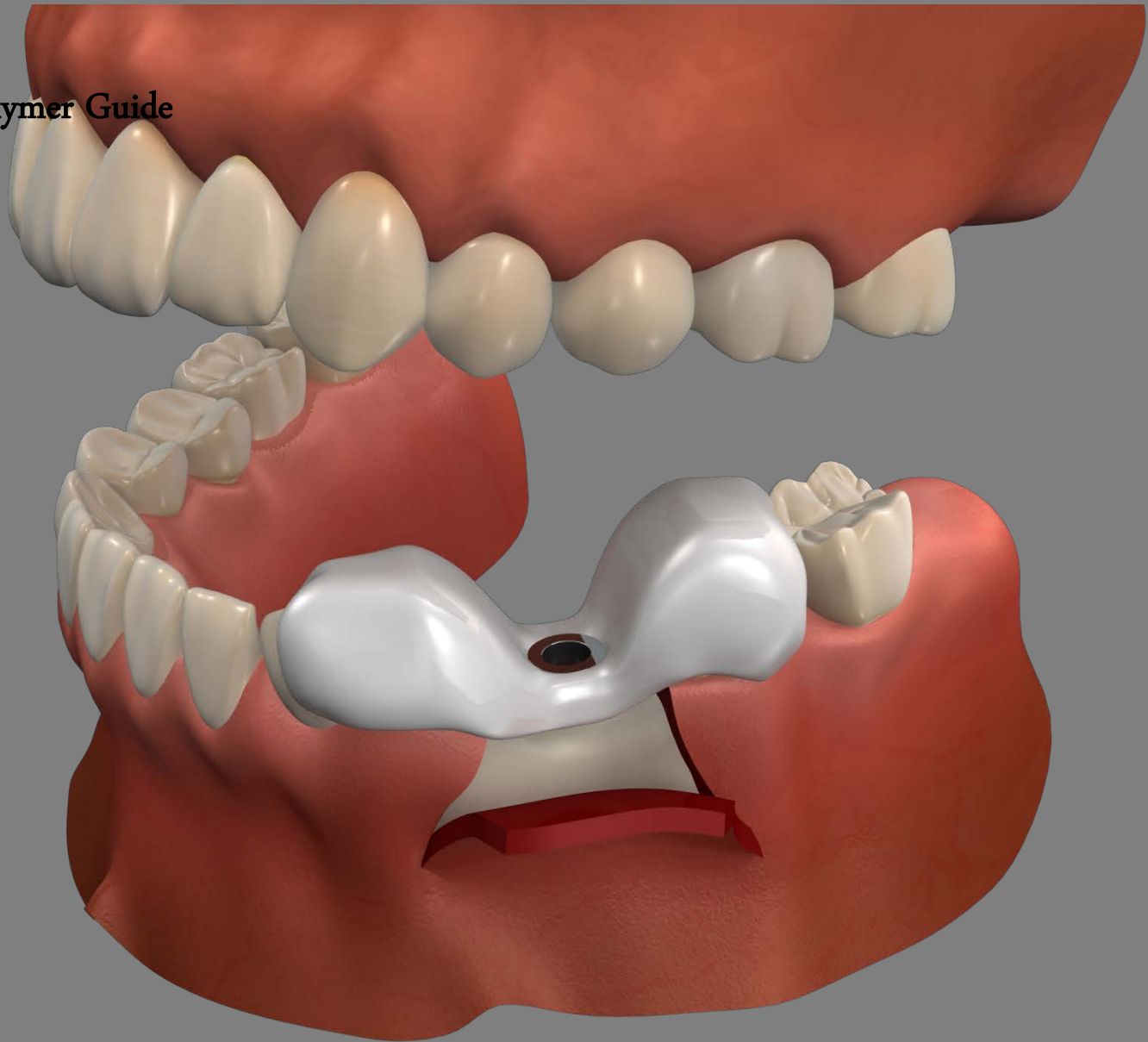




**Position the Polymer Guide  
in the mouth**

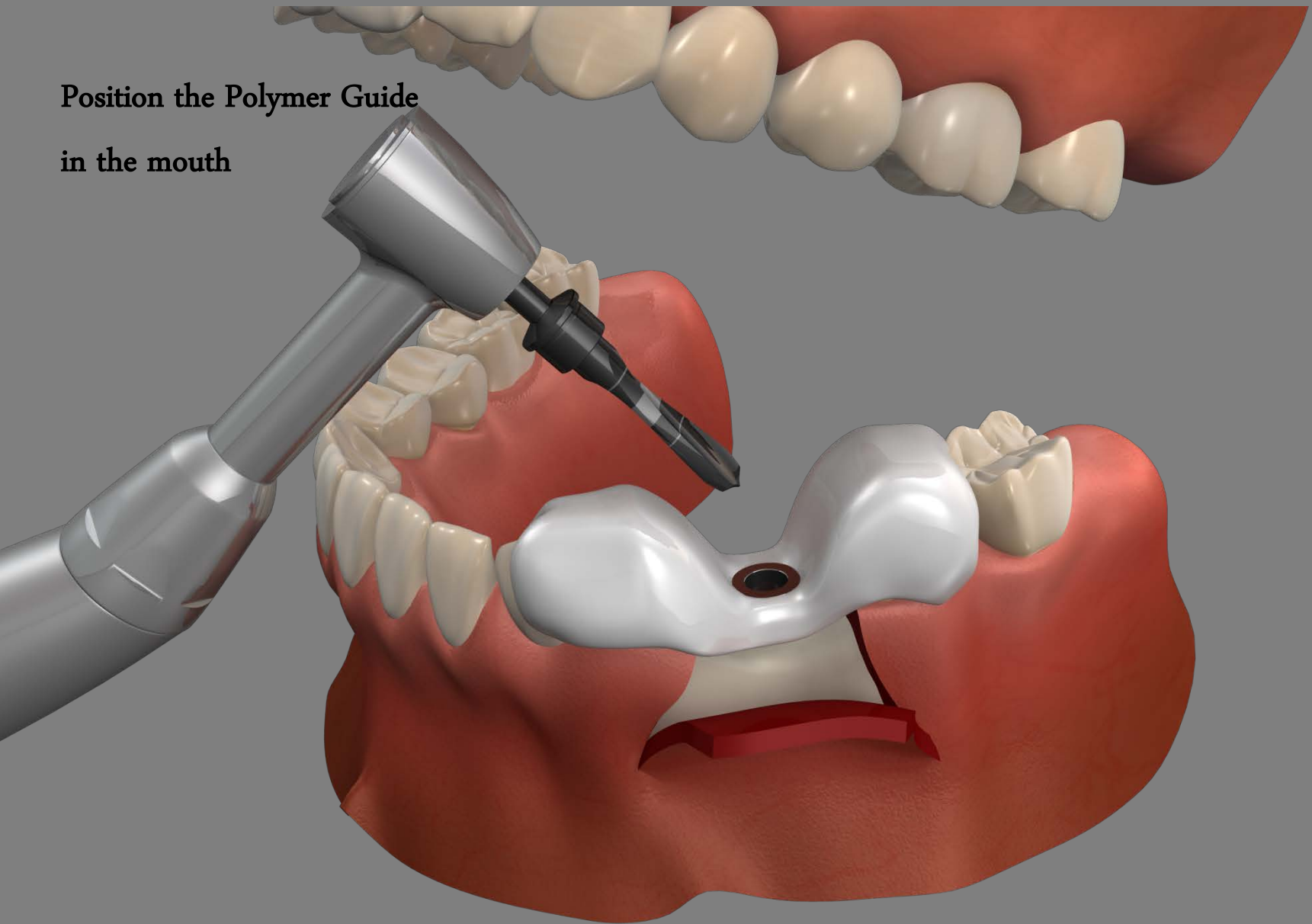





**Harvest drill - XFH 3435**

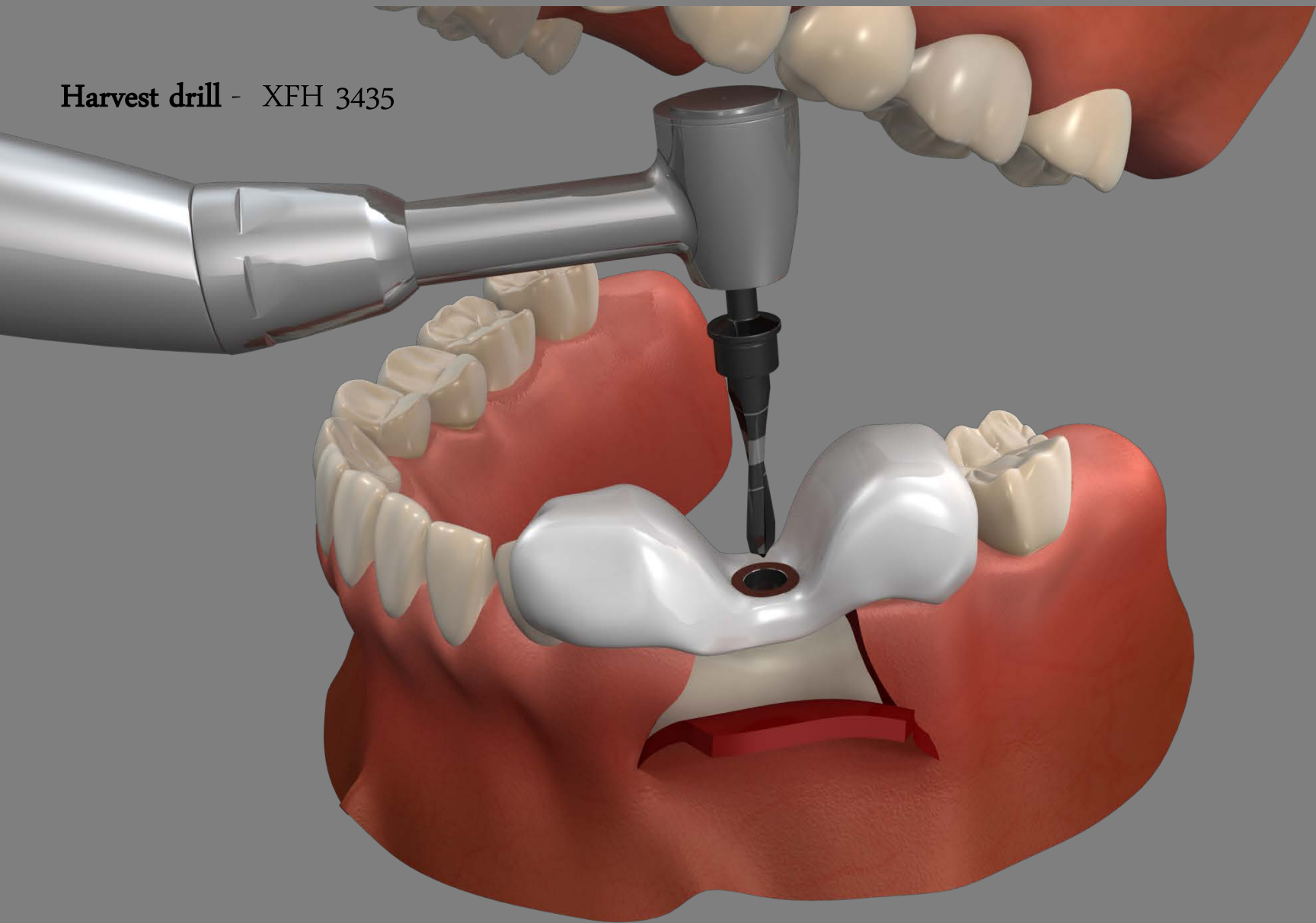


**Harvest drill** - XFH 3435

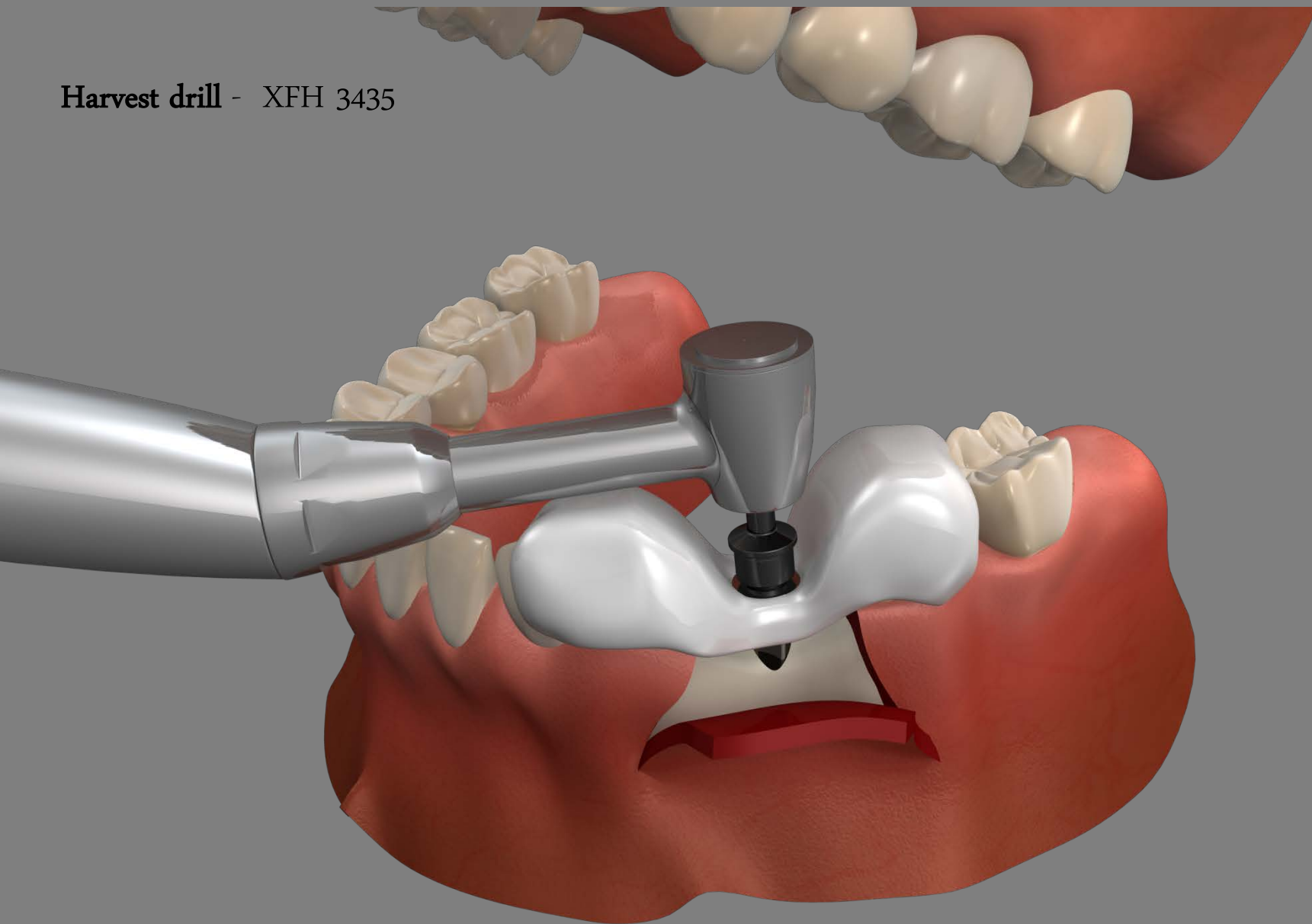

**Harvest drill - XFH 3435**

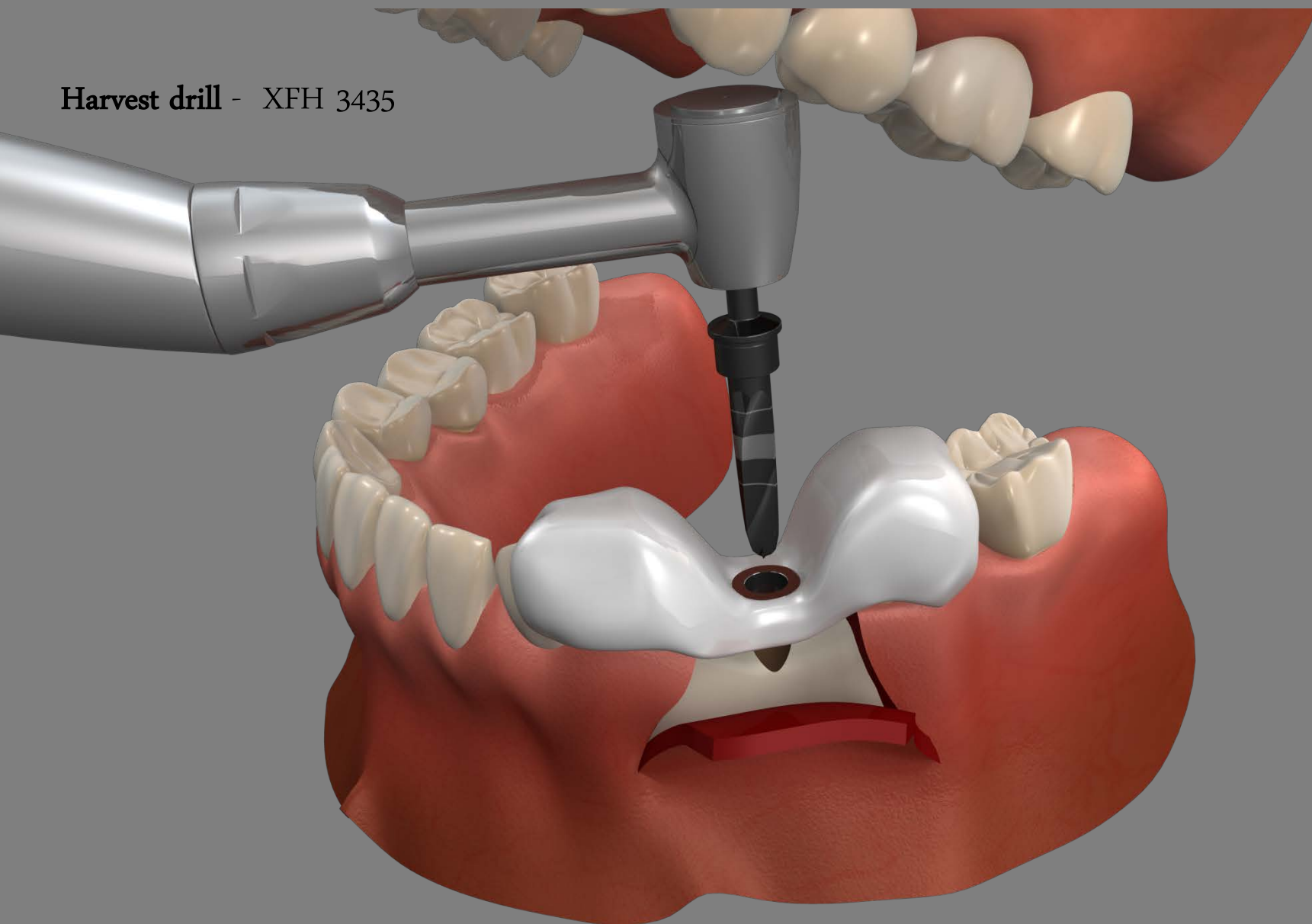

## Removal of Polymer Guide

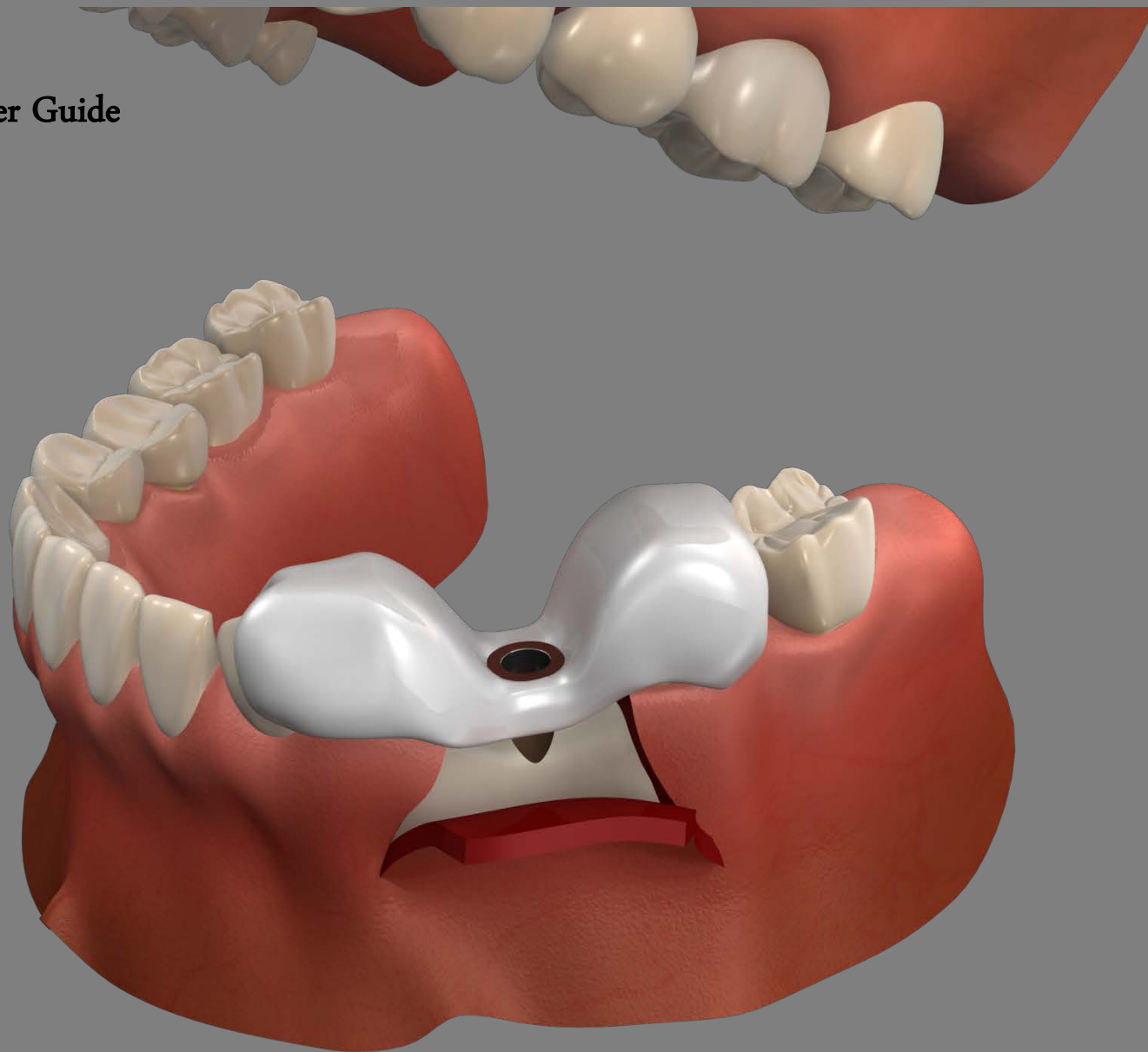

## Removal of Polymer Guide

